CORMAY CYSTATIN C CONTROLS

CORMAY CYSTATIN C CONTROLS

- Nr kat. 4-460 2 x 1 ml
• CYSTATIN C CONTROL (L) 1 x 1 ml
• CYSTATIN C CONTROL (H) 1 x 1 ml

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INTENDED USE

CORMAY CYSTATIN C CONTROLS are designed for quality control of cystatin C determinations at human serum and plasma.

The reagents must be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

INTRODUCTION

Cystatin C is a low molecular weight protein (13kD), one of the cysteine proteinases inhibitors. Cystatin C is produced in all nucleated cells and secreted into extracellular space at constant rate. Cystatin C molecule stability and dependence of its concentration solely on GFR (Glomerular Filtration Rate) decide about high diagnostic efficiency of cystatin C determination. Cystatin C level is not affected by muscle mass or diet and its increase is observed even at slight GFR reduction. Clinical applications of cystatin C are for monitoring GFR in children, elderly patients, patients with potentially nephrotoxic drug therapy, for assessment of renal transplantation status, for kidney function monitoring in acute and chronic kidney diseases including diabetic nephropathy.

STABILITY AND STORAGE

- Controls are ready to use.
- Unopened controls are stable at 2-8°C until the expiration date printed on the package.
- Controls after opening are stable up to expiration date, if stored in refrigerator thoroughly closed.
- Do not allow controls to stand for long time periods at room temperature.
- Do not freeze controls!

WARNINGS AND NOTES

- Do not use calibrators after expiry date.
- Serum used in the manufacturing of control are tested for HBsAg, anti-HCV, anti-HIV1 and anti-HIV2 and found to be negative. Nevertheless, the materials contain substances of human origin and must be handled with due care.
- Controls contain antibiotics as a preservative and must be handled with due care.

ASSIGNED VALUES

CONTROL	Lot No	TARGET	RANGE
L	807-20	0.99 mg/l	0.79 – 1.19 mg/l
H	807-20	3.42 mg/l	2.74 – 4.10 mg/l

WASTE MANAGEMENT

Please refer to local legal requirements.

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