CORMAY FIBRINOGEN CALIBRATOR

CORMAY FIBRINOGEN CALIBRATOR (PL)
Nr kat. 4-292 1 x 1 ml

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CORMAY FIBRINOGEN CALIBRATOR (EN)
Cat. No 4-292 1 x 1 ml

INTENDED USE

CORMAY FIBRINOGEN CALIBRATOR (lyophilisate) is designed for use as a calibrator for determination of fibrinogen concentration. Use in accordance with instructions enclosed to reagents for determination of fibrinogen concentration.

The reagents must be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

CHARACTERISTICS

The CORMAY FIBRINOGEN CALIBRATOR is a human biological fluid containing human fibrinogen. Calibrator was processed from human plasma collected with sodium citrate anticoagulant. Stabilisers are added before lyophilisation.

RECONSTITUTION

Carefully remove the rubber cap from vial and add 1 ml distilled water to the vial. Recap, swirl gently and let stand undisturbed for 15 minutes at room temperature. Do not invert vial or mix vigorously.

PROCEDURE

Before analysis calibrator and samples should be diluted 20-fold with 0.9% NaCl and mixed gently. In case of fibrinogen analysis in HITACHI 911/912 analyser, dilution of calibrator and samples is not required.

STABILITY AND STORAGE

Unopened calibrator is stable at 2-8°C until the expiration date printed on the vial.

Once reconstituted, calibrator is stable for 8 hours when stored at 2-8°C. Gently mix contents before each use.

NOTES

- The calibrators are based on human serum that has been tested for HBsAg and antibodies to HIV and HCV and found to be non-reactive. However, this material should be handled as thought capable of transmitting infectious disease.

ASSIGNED VALUES

Parametr	Lot No	Calibrator Value
Fibrinogen	002-36	247 mg/dl

Calibrator values (Lot No 002-36) for fibrinogen determination in HITACHI 911/912 analyser:

Calibrator	Concentration
0	0 mg/dl (deionized water)
1	39.5 mg/dl
2	98.8 mg/dl
3	198 mg/dl
4	395 mg/dl
5	593 mg/dl

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