CORMAY HDL/LDL CALIBRATOR

CORMAY HDL/LDL CALIBRATOR (PL)
Nr kat. 5-178 1 x 1 ml

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CORMAY HDL/LDL CALIBRATOR

CORMAY HDL/LDL CALIBRATOR (EN)
Cat. No 5-178 1 x 1 ml

WASTE MANAGEMENT
Please refer to local legal requirements.

Date of issue: 09. 2020.

INTENDED USE

Calibrator for the determination of HDL/LDL-cholesterol concentration by direct method.

The calibrator must be used only for *in vitro* diagnostic, by suitably qualified laboratory personnel, only for the intended purpose, under appropriate laboratory conditions.

INTRODUCTION

Plasma lipoproteins are spherical particles containing varying amounts of cholesterol, triglycerides, phospholipids and proteins. The relative protein and lipid determine the density of these lipoproteins and provide the basis on which to begin their classification. The classes are: chylomicron, very-low-density lipoprotein (VLDL), low-density-lipoprotein (LDL) and high-density lipoprotein (HDL). The principle role of HDL in lipid metabolism is the uptake and transport of cholesterol from peripheral tissues to the liver. Low HDL cholesterol (HDL-C) levels are strongly associated with an increased risk of coronary artery disease.

CORMAY HDL/LDL CALIBRATOR is lyophilised human serum intended as a standard for the determination of HDL or LDL-cholesterol by direct method.

RECONSTITUTION

Reconstitute the contents of CORMAY HDL/LDL CALIBRATOR vial with exactly 1 ml of distilled water, close the vial and let it stand for 30 minutes. Mix carefully to ensure homogeneity of the solution. Avoid foaming.

STABILITY AND STORAGE

Unopened calibrator is stable at 2-8°C until the expiration date printed on the vial.

Stability after reconstitution:

- at 2-8°C – 5 days
- at -18°C to -24°C:
- ✓ 21 days (for LDL assay)
- ✓ 30 days (for HDL assay)

WARNINGS AND NOTES

- Calibrator may be frozen only once.
- The calibrator is based on human serum that has been tested for the HIV antibody, HbsAg and HCV and found to be non-reactive. However this material should be handled as thought capable of transmitting infectious disease.

ASSIGNED VALUES

| CORMAY HDL/LDL CALIBRATOR | | Lot: 009-55 | |
|---------------------------|-----------------------|---------------|--------|
| Component | Method | Concentration | |
| | | mg/dl | mmol/l |
| HDL | Direct, II generation | 66.2 | 1.72 |
| LDL | Direct, II generation | 140 | 3.63 |

TRACEABILITY

HDL/LDL calibrator is traceable to the NIST: SRM 1951c reference material.

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